



## SHOWER SEATS AND HAND RAILS

The elastic parts of the Sense and Gaius shower seats and support rails are made of PUR-IMC integral plastic. With appropriate care, the long service life of the material can be extended further. The metal parts of the products are painted with antimicrobial paint.

**Cleaning:** Remove any discharge immediately with a disinfectant cleaner. The disinfecting agent kills microbes and prevents their growth. These cleaners contain surfactants, a disinfecting agent and alkalis or acids. They may be hydrogen peroxide-based.

Daily cleaning is the best way to avoid persistent stains and discolouration. Use neutral or slightly alkaline (pH 7 - max. 10.5) all-purpose cleaner for maintenance cleaning. The cleaning product must not form layers on the surface of an antimicrobial product. Therefore, it is important to choose the correct cleaning products and follow the dosage instructions. Do not use abrasive cleaning products or equipment. Microfibre cloths are recommended because of their efficient mechanics.

For more information, please visit [www.korpinen.com](http://www.korpinen.com)

